

ESSENTIAL ADVANTAGES OF THE COMBIFIX-TUBE BAG



Easy handling

The bag system and Irrigation cone fit perfectly – incorrect operation is therefore impossible.



Generously dimensioned

Combifix has a sufficient length so that the tubular bag can also be used for irrigation standing up.



Hygienic and clean

The closed bag design ensures that

- No splashes get outside
- Your hand does not get in contact with excreta
- Only a very slight odour nuisance is produced



High quality production

The pressure plate is firmly welded to the bag. So do not worry about torn connections or leaking joints.

For more information, see:

www.schaefers-medizintechnik.com



**WE OFFER COMPETENT
CONSULTING AND
ADVICE –
CALL OR WRITE TO US.**

combifix

CONTACT DATA:



**Schaefers
Medizintechnik**

Postfach 2115
46342 Velen
Telefon: +49(0)2863 95024

service@schaefers-medizintechnik.com
www.schaefers-medizintechnik.com


combifix

COMBIFIX AN EXCEPTIONAL TUBULAR BAG


As of 2023-02-27

CLEAN, SAFE, FEEL-GOOD


COMBIFIX TUBULAR BAGS FOR IRRIGATION

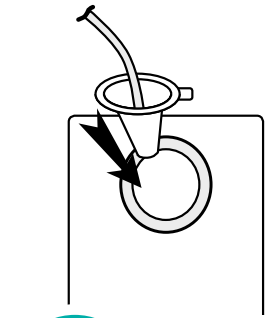


1 ATTACH CONE
snap the cone into the snap ring

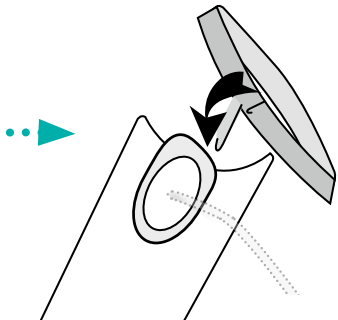

2 CLIP ON BELT
snap in the belt on the right and left

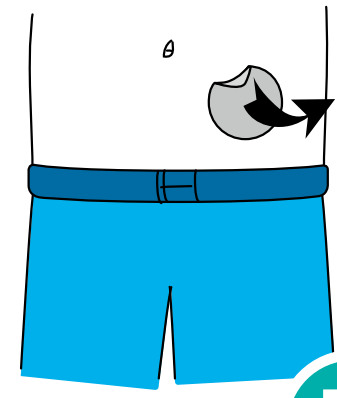


3 OPEN STOMA
remove old supply from stoma

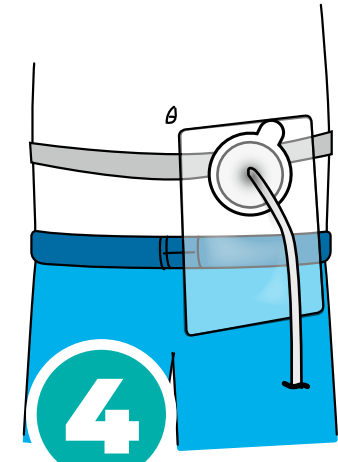

4 ATTACH BAG
place the bag over the stoma on the abdomen


CLOSE THE BAG

after the water has run in, remove the cone and press the lid onto the opening and snap it into place. Finally, fold up the bag and close it with the clip.

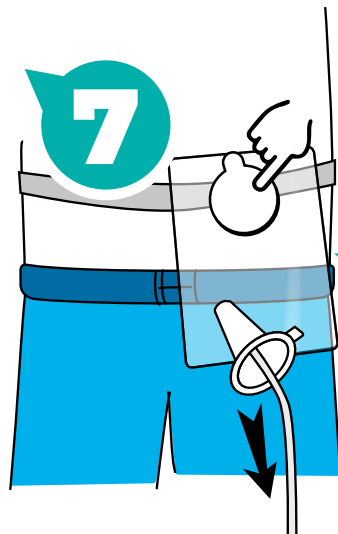


7

8 DISPOSAL
after irrigating, dispose in the residual waste.

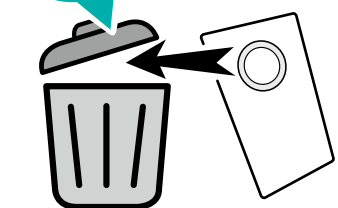

6 IRRIGATE
switch on the irrigated and let water run into the stoma (irrigate)

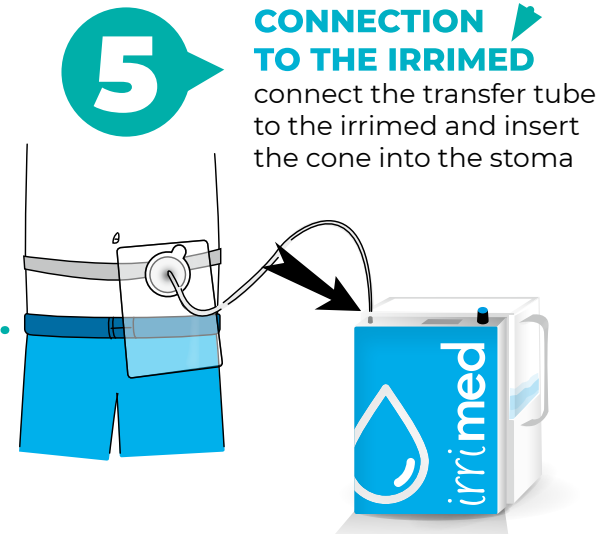

5 CONNECTION TO THE IRRIGATED
connect the transfer tube to the irrigated and insert the cone into the stoma