

HOW TO USE IRRIMED?

The application is rather easy because it is similar compared to conventional cleaning/rinsing methods.

Fill your irrimed with nicely tempered tap water, which is then pumped via a transfer tube through a cone or rectal catheter into the rectum.

This stimulates a stretch, which in turn activates the peristaltic and therefore also a bowel emptying.

Consequently, the rectum and a part of the colon have been emptied, allowing you to achieve stool-free periods of 24 hours and more.

You can find a detailed description of the process in our instruction manual, see:

www.schaefers-medizintechnik.com



OUR SERVICE PLEDGE

- **Competent counselling**
- **Repairs in less than a week**
- **Free replacement units**

Order details:

irrimed stoma
Order number: 5000

Irrimed rectal and all needed accessories are available at specialist dealers or for self-paying customers directly with us.
Irrimed rectal can be prescribed by prescription.

A PRODUCT BY:



Schaefers
Medizintechnik GmbH

Im Wiesengrund 13
46342 Velen
Telefon: +49(0)2863 95024

service@schaefers-medizintechnik.com
www.schaefers-medizintechnik.com

irrimed classic

irrimed classic

THE MODERN SOLUTION FOR RECTAL IRRIGATION



As of 2024-02-06

CLEAN, SAFE, FEEL-GOOD

RECTAL IRRIGATION WITH THE IRRIMED

WHAT IS RECTAL IRRIGATION?

The best and most efficient way to regain control over bowel emptying after losing it, due to neuraxial paralysis or sphincter deficiency, is rectal irrigation.

Fecal incontinence and other issues regarding bowel emptying are now in the past.

Irrimed rectal offers more security and allows you to enjoy your private and social life.

Additionally, it has a positive impact on the environment by using fewer resources than conventional methods.



INDEPENDENT

Thanks to its modern and powerful Li-Ion battery, irrimed rectal can be used anywhere, anytime, autonomous of the mains supply

COMFORTABLE

You can easily check the water temperature from the large LCD display.

COMPACT

While travelling simply remove the handle and safe space.

MOBILE

The ergonomically optimal water container and separate handle allow you to carry the full container anywhere you like, with ease.

SECURE

You can fully recharge the battery in max. 8 hours, with the quick charger, which can be used world-wide.